


b e n t e c

STERILE

Silicone Biliary Stent

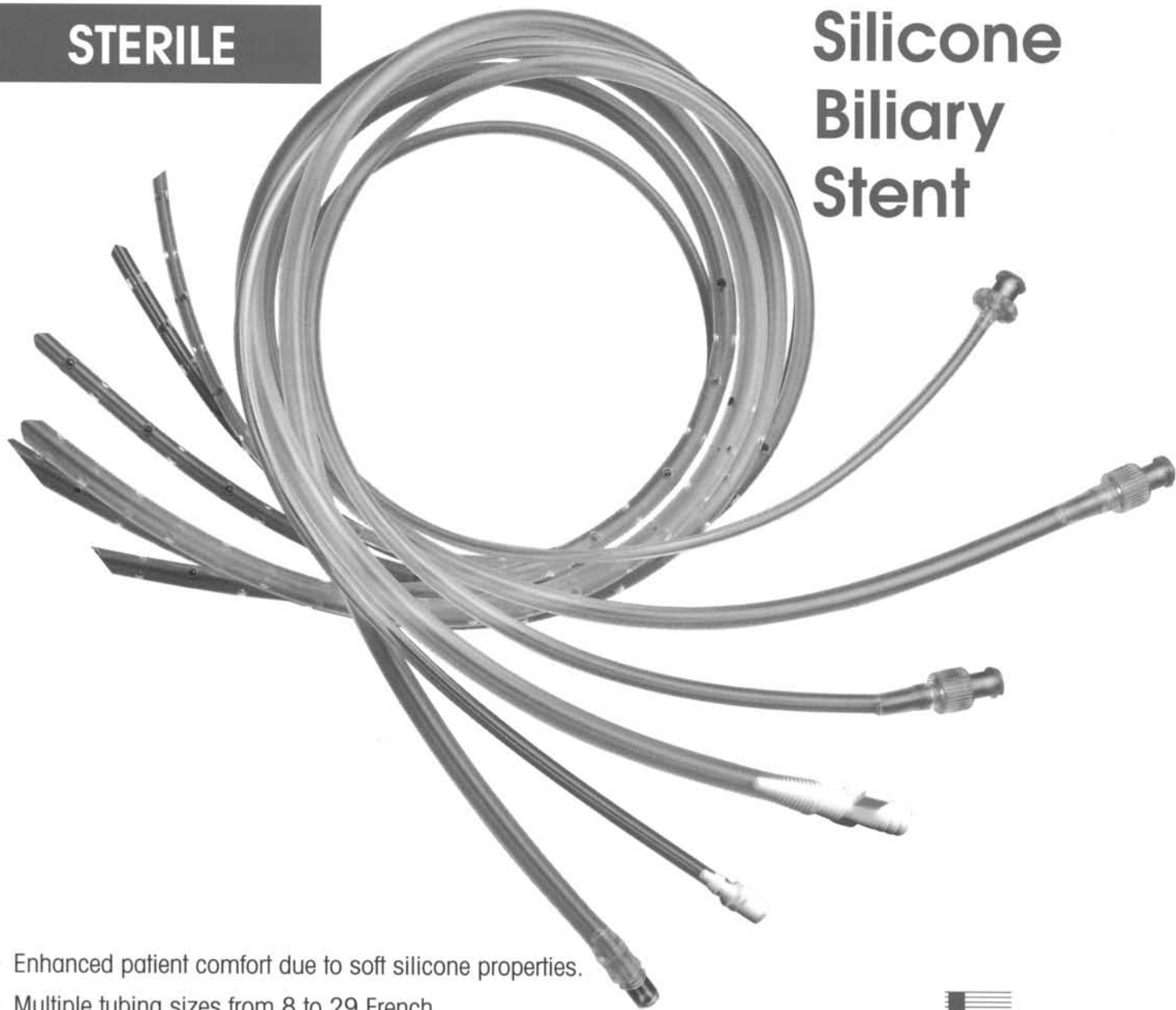

- Enhanced patient comfort due to soft silicone properties.
- Multiple tubing sizes from 8 to 29 French.
- Can be cut to preferred lengths.
- Up to 36 holes for efficient drainage.
- Radiopaque marker for x-ray visualization and positioning.
- Includes luer lock adapter to facilitate percutaneous drainage connection.

To order, call 800•767•9175

Visa/Mastercard/Procurement Cards Accepted



**1380 E. Beamer St.
Woodland, CA 95776**

**Fax: 888-828-8869
Internet: www.bentecmed.com**

Silicone Biliary Stent

- Biocompatible material for long-term placement.
- Soft consistency provides for patient comfort.
- Easily changed percutaneously.

The following procedure is provided by Dr. Malonnie Kinnison and Dr. Robert White*:

Introduction of the Silicone Biliary Stent

A tract is first formed with an 8.3 or 10 Fr. polyethylene pigtail drainage catheter by standard techniques. After the tract has matured, a 0.38 HSF torque guide wire is placed through the pigtail catheter to maintain access to the biliary tract, and the pigtail is removed. A conventional Staple-van Andel catheter of appropriate size is lubricated with sterile lubricant and then placed through the silicone stent tube so the tapered tip extends 5-6 cm beyond the end of the tube. The stent tube and catheter are also externally coated with sterile lubricant to facilitate passage through the

liver and reduce patient discomfort. Due to the soft nature of the tube and the relatively thin wall, a slow, firm and forward rotary motion of the coaxial system will facilitate in successful placement. The tube may otherwise accordion at the skin and not advance forward. When the tip of the stent is in the duodenum or small bowel the coaxial catheter and guide wire are removed. The location of the proximal side hole within the intrahepatic ducts is confirmed by noting the radiopaque marker within the ducts on a post-insertion cholangiogram.

*Dr. Malonnie Kinnison, Dr. Robert White. The Johns Hopkins Medical Institutions.
The Russell H. Morgan Department of Radiology and Radiological Sciences
Baltimore, MD 21205



Silicone Biliary Stent

STERILE

Product Code	Size
GR83064-12	8 Fr.
GR83064-03	10 Fr.
GR83064-01	12 Fr.
GR83064-02	14 Fr.
GR83064-04	16 Fr.
GR83064-05	18 Fr.
GR83064-08	19 Fr.
GR83064-06	20 Fr.
GR83064-07	22 Fr.
GR83064-10	24 Fr.
GR83064-13	26 Fr.
GR83064-14	28 Fr.
GR83064-11	29 Fr.

All Bentec products are latex-free

To order, call 800•767•9175

Visa/Mastercard/Procurement Cards Accepted • Special Orders Available

Caution: Federal (USA) law restricts these devices to sale or on the order of a physician.
All Bentec Medical, Inc. products are manufactured in the U.S. by Bentec Medical, Inc. in compliance with strict GMP Guidelines.



1380 E. Beamer St.
Woodland, CA 95776
Fax: 888-828-8869

Internet: www.bentecmed.com
E-mail: bentec@bentecmed.com